


## -DRAPING INSTRUCTIONS-

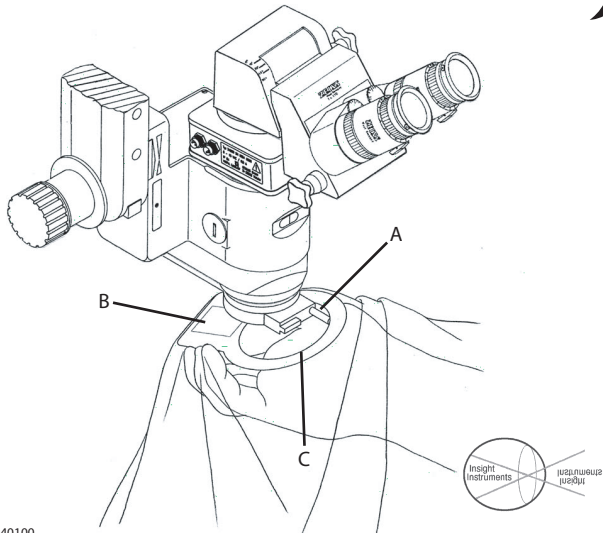

1. Place sterile BION adapter onto microscope and rotate until "pin" faces forward. (A)
2. On drape, locate horizontal fold and unfold up.
3. Eye piece covers should now be exposed facing nurse.
4. Place hand in lower cuff and grasp cardboard collar.
5. Place hand in upper cuff and open drape.
6. Remove backing from adhesive sticker. (B)
7. Angle cardboard collar over the front of adapter (C) and press adhesive area of collar up against back of adapter.
8. Drape over head of microscope and position eye piece covers
9. Unfold drape back to post.
10. Remove perforation from white tie straps, wrap around "arm" of scope and adhere to drape.
11. Snap off the perforated tab on eye piece covers.
12. For BION 3e/3c systems, locate small white stickers with arrows 6-10" below central eyepieces and pull apart arrows for insertion of BION drive shaft (3e system) or BION motor cable (3c system).